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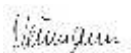
O-Phosphoric acid 37% (acc. 85% 0.395g)

3. Indications

Etching of Enamel / Using by adhesiv dentistry

4. Contraindication

Placement of C-Cid is contraindicated if the patient is known to be allergic to any of the ingredients of C-Cid.

5. Side Effects

Application of eugenol containing material, specifically base liners, is not indicated, it will impair the polymerization.

6. Processing and Placement

-Tooth cleaning

Clean the tooth to be treated and the adjacent distal surfaces with a bristle brush and prophylactic paste. Clean the interdental spaces with a dental floss.

-Cavity Preparation

The cavity is prepared in the standard manner. Undercuts and bevelled margins are recommended to improve adhesion between tooth and filling and to optimize marginal adaption.

In the occlusal area the rules of amalgam preparation are applied.

-Insulating

To achieve the best results it is absolutely essential that the tooth is thoroughly dry. The use of a rubberdam is therefore recommended.

-Base Liner

To protect exposed dentine the application of a glass ionomer liner, phosphate cement or calcium hydroxide base liner (deep cavities) is recommended, thus is reducing the danger of secondary caries at the margins.

-Acid Etching

The cleaned enamel edges are etched with C-Cid. C-Cid is applied with the supplied brush. After 30-45 s C-Cid is applied on the dentine allowed to take effect for another 10-15 s and then thoroughly rinsed with water. Excessive drying should be avoided in this case, as this would cause the exposed collagen fibre to collapse

6. Shelf life

3 years

7. Precautions and Storage

The etching contains 35% phosphoric acid. Contact with oral soft tissues, skin and eyes must be avoided.

Not to be stored above 25°C . Storage in refrigerator prolongs shelf life

8. Deliver units

C-Cid	single syringe	3.0 g
	part of assortment packages	

9. Manufacture and Sales

MEGADENTA Dentalprodukte GmbH

D-01454 Radeberg, Germany, Tel. 03528 453-0, Fax: 03528 453-21

www.megadenta.de / info@megadenta.de