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11.01.2016 /dt.-engl.

Megaseal N properties and advantages:

- **easy and fast application**
- **high fitting accuracy**
- **high scratchable resistance**
- **high gloss oversize**
- **cureable in all dental light units**

1. Product description

Megaseal N is a light curing lacquer for the coating of resin parts of acrylic dentures as well for temporaries in C+B technique. A high abrasion-resistant and high gloss polished surface can be produced.

2. Application

Megaseal N can be applied with a brush directly onto the oversize of the denture or temporary. For safe adhesion, the surface should be free of grease and dry.

The layer should be thin. If a thicker layer is required, the application should be in 2 steps with each separate curing process.

The curing process should be effected in a light-polymerization unit with UV- or Blulight spectrum (ea. Megalight) with a wavelength of 320-500 nm. Based on the intensity of the unit, the curing time should be 3-5 minutes.

3. Technical data / Important advice

Attention! Megaseal N contains Methylmetacrylate!
It is flammable liquid. Keep away from fire or heat.
Do not inhale and do not get in contact with skin or eyes.

Contains: Methylmethacrylate, Polyacrylate, Photoinitiator, Stabilisator, Nanoparticles (SiO₂ base)

4. Shelf life

2 years

5. Package and delivery units

20 ml bottle with brush insert
30 ml bottle with drop insert

Art.-Nr. 2VLACK-N2
Art.-Nr. 2VLACK-N

6. Date information prepared

11.01.2016 /dt.-engl.