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Carl-Eschebach-Str. 1A / D-01454 Radeberg, Germany

Instructions for use

1. Name and description of medical device product

C-Cid

Etching gel for etch&rinse technique

Shade: blue

2. Content

C-Cid gel contains: o-Phosphoric acid 37 wt.% , thickening agent and colour pigments

3. Indications

C-Cid is used for enamel etch or etch&rinse technique for:

- composite restorations
- sealing of fissures

4. Contraindication

C-Cid must not be applied to acid-soluble underfillings.

5. Side effects

Not known to date.

6. Interactions

Not known to date.

7. Hazard Information

Read Safety data sheet first!

Wear protective gloves/protective clothing/eye protection/face protection.

After CONTACT WITH SKIN, wash immediately with polyethylene glycol, followed by plenty of water. Take off immediately all contaminated clothing and wash it before reuse. If skin irritation occurs: Get medical advice/attention. Immediate medical treatment required because corrosive injuries that are not treated are hard to cure.

In case of CONTACT WITH EYES flush immediately with plenty of flowing water for 10 to 15 minutes holding eyelids apart and consult an ophthalmologist.

Wash hands thoroughly after handling. Close the syringe after each use.

8. Handling

-Enamel Etching

Once the cavity has been set up and, if necessary, the underlay is laid and cured, the cleaned melt edges are etched with C-Cid. Apply C-Cid with the enclosed brush and rinse with plenty of water after a contact time of 40-60 s (for teeth of the 1st dentition: 80-120 s). Subsequently, it must be dried carefully. The etched and dried enamel must have a matt, chalky surface. The etched enamel must not be contaminated by saliva or blood. This leads to the inactivation of the etched enamel edges due to the addition of proteins. For this reason, it is essential to ensure absolute drainage (Kofferdam). If the etched enamel is contaminated nevertheless, the etching process should be repeated.

- Enamel and dentine etching (total etch technique)

If dentin adhesives are used, the enamel and dentin should be etched. The smear layer of the dentin is not removed and the unprepared enamel is cleaned with a cleaning paste. The cleaned enamel edges are etched with C-Cid as usual. C-Cid is applied with the enclosed brush. After an exposure time of 30-40 s (for teeth of the first dentition: 60-80 s) C-Cid is additionally applied to the dentin, without rinsing with water. After 10-15 s, rinse thoroughly with plenty of water. It must not be excessively dried. This would cause the exposed collagen fibers to collapse. The dentin adhesive is applied according to the manufacturer's instructions.

9. Shelf life

Date of expiration: see note on syringe/packaging.

10. Precaution and storage

Not above 25 °C, Refrigerated storage (+ 4 °C to + 8 °C) is recommended.

11. Delivery units

3 g / 5 g syringe in single pack and as a component of assortment packs

Luer-Lock Cannulas

12. Waste treatment

Do not allow to enter into surface water or drains. Do not allow to enter into soil/subsoil. Dispose of waste according to applicable legislation. Handle contaminated packages in the same way as the substance itself.

13. Date information prepared

2018-04-04

14. Manufacturer and address

MEGADENTA Dentalprodukte GmbH
Carl-Eschebach-Str. 1A / D-01454 Radeberg, Germany