C-Prime S Plus

[de]	Gebrauchsanweisung
[bg]	Инструкции за употреба
[cs]	Návod k použití
[en]	Instructions for use
[es]	Instrucciones de uso
[fr]	Mode d'emploi
[hu]	Használati utasítás
[hr]	Upute za uporabu
[it]	Istruzioni per l'uso
[lt]	Naudojimo instrukcija
[lv]	Lietošanas pamācība
[pl]	Instrukcja użytkowania
[ro]	Instrucțiuni de utilizare
[ru]	Инструкция по применению
[sk]	Návod na použitie
[sv]	Bruksanvisning

MEGADENTA
Dentalprodukte

MEGADENTA Dentalprodukte GmbH
Carl-Eschebach-Str. 1A, D-01454 Radeberg, Germany
Telefon +49(0)3528 453-0, Fax +49(0)3528 453-21
E-Mail: info@megadenta.de www.megadenta.de

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[en] Instructions for use

Read and keep before using the product! Medical device for professionals only.

1 General safety instructions

Warning! C-Prime S Plus contains the photoinitiator EHA (2-ethylhexyl-4-dimethylaminobenzoate), a hazardous substance classified as a CMR substance under Regulation (EC) No. 1272/2008. When handling this product, the protective measures listed below must be strictly observed. When light polymerization is performed correctly, EHA is largely consumed; residual levels in the polymerized product are very low and, based on current knowledge, fall within a range that is toxicologically safe.

Avoid contact of uncured material with skin/mucous membrane and eyes. It may be slightly irritating and in rare cases may cause sensitization to methacrylates.

Use protective equipment: Safety glasses, suitable protective gloves with protection against Methacrylates (e.g. Polyethylene-gloves, 4h-gloves).

After skin contact: Wash immediately with plenty of soap and water. Medical treatment necessary if symptoms occur (irritation of the skin).

After eye contact: Immediately flush with running water for 10 to 15 minutes with the eyelid open and consult an ophthalmologist.

If swallowed: Rinse your mouth immediately and drink plenty of water.

In the course of the service life of a filling cemented with a bonding agent, which may be several years, the possibility of chemicals/substances being dissolved out of the resin matrix cannot be completely ruled out. This depends on various factors (e.g. nutritional habits of the patient). The patient must be informed about the residual risk of genotoxic, cytotoxic and/or sub-acute systemic toxic effects, with a special focus on pregnant women, nursing mothers and children.

The publicly available Summary Safety and Clinical Performance Report (SSCP) according to Article 32 of Regulation (EU) 2017/745 (MDR) can be found in the EUDAMED database at the following link: <https://ec.europa.eu/tools/eudamed/#/screen/search-device>

All serious incidents related to the product must be reported to the manufacturer and the competent authority of the member state where the user and/or patient is established. Please direct any feedback regarding the product to: **reclamation@megadenta.de**

2 Description

2.1. General description

C-Prime S Plus is a polymer-based, self-adhesive fixing material class 2, classification according to DIN ISO/TS 16506:2021

C-Prime S Plus is a fluid, light-curing, primer intended for use in combination with an adhesive on tooth structure (enamel/dentin).

2.2. Composition

Ethanol, HEMA, TEGDMA, 1,3-glyceryl-DMA, bis-GMA, methacrylate-modified maleate oligomer, camphorquinone / EHA (0.2% each), TPO / BHT (0.1% each)

3 Intended use

3.1. Warnings

C-Prime S Plus is light-sensitive, so that polymerization may already be triggered by exposure to daylight or by ambient light from the dental treatment lamp. The material should therefore be removed from the primary packaging only immediately before application and the packaging resealed directly. In case of damage to the primary packaging (bottle), the product must be disposed in accordance with 3.12.

3.2. Intended purpose / Intended use

For use in combination with an adhesive for:

- Bonding to enamel/dentin
- Intraoral repair of composite restorations

The user is obliged to use the material in accordance with the intended purpose specified by the manufacturer MEGADENTA Dentalprodukte GmbH. The manufacturer accepts no liability for damage resulting from use for other purposes.

3.3. Clinical benefit

The clinical benefit of C-Prime S Plus lies in its ability to help preserve or restore the function and aesthetics of affected teeth.

3.4. Indications

- Loss of tooth structure
- Defects in composite restorations

3.5. Contraindications

- insufficient moisture control of the working area
- known hypersensitivity to methacrylates
- eugenol-containing underfillings (see 3.9)

3.6. User profile

C-Prime S Plus is a product for professional use by dentists. Application is intended only in a dental practice. Special training is not required.

3.7. Patient group

The use of C-Prime S Plus is not limited to any particular patient group. Contraindications and safety instructions must be observed.

3.8. Undesirable side effects

C-Prime S Plus contains methacrylates. These may cause allergic skin reactions. In rare cases sensitization is possible. Pulpal irritation cannot be excluded in the case of cavities close to the pulp. A protective base/ underfilling is recommended in such cases.

3.9. Undesirable Interactions

C-Prime S Plus must not be used together with eugenol-containing underfillings, as phenolic substances inhibit polymerization.

3.10. Special storage instructions

Store protected from light and at 4-25 °C.

Storage outside the specified conditions may result in damage to the product, e.g. worse mechanical performance and reduced applicability. It can then no longer be applied as intended and must be disposed of in accordance with 3.12.

3.11. Shelf life

2 years. Do not use after the expiry date stated on the label!

3.12. Waste treatment

Do not allow product or primary packaging which has not been emptied of all residues to reach sewage system, water, subsoil or soil. Disposal according to official regulations.

Non-contaminated and completely empty packaging as well as fully cured products can be disposed of with normal municipal waste (household waste). Observe waste separation.

4 Application

4.1. Drying

Thoroughly dry the surfaces to be treated. The use of cofferdam is recommended.

4.2. Cavity preparation

Cavity preparation is carried out according to the rules of the adhesive filling technique. In the anterior region, bevel the enamel margins; in the posterior region, only slightly break or round off the sharp enamel edges, always sparingly and conserving the substance. Then remove all residues and dry the cavity.

4.3. Underfilling

If an underfill is to be placed (e.g. for cavities close to the pulp), make sure that the material is acid-resistant and eugenol-free.

4.4. Preparation for Adhesion

- Tooth structure: Apply the etching agent according to the manufacturer's instructions. Carefully dry the etched dentin with a clean, oil-free air stream. It must not be allowed to dry out.
- Existing composite restoration: Roughen the surface.

4.5. Application

- Allow material to acclimatize to room temperature before use.
- Do not use the bottle inside the mouth to avoid cross-contamination! Drip out C-Prime S Plus into a presentation dish (1-2 drops depending on the cavity size) and close the bottle.
- Apply the primer to the enamel and dentin using a brush or mini sponge and massage it in for at least 15 seconds. The dentin surface should always be moist during this time.
- Carefully blow off the layer with oil-free air for 10 s to evaporate the solvent (ethanol). The dentin must not dry out in the process.
- Polymerization is performed (where applicable, only after the adhesive has been applied) using a commercially available blue light source, e.g. Megalux CS (emission wavelength range 400-500 nm, irradiance 600-800 mW/cm²).
- The exposure time with the recommended device is 10-20 s.
- The instructions for use of the device used must be observed!

4.6. Application of the adhesive

Next, apply the adhesive according to the manufacturer's instructions. Megadenta recommends using N-Bond.

4.7. Finishing

The material to be bonded (e.g., restorative material) can now be applied according to the manufacturer's instructions. C-Prime S Plus with N-Bond is compatible with all restoration materials from MEGADENTA Dentalprodukte GmbH.

5 Delivery units

Article no.: 2PRIME-S bottle 5 ml

6 Symbols used in product labeling

	Manufacturer
	Medical device
	Item number
	Batch
	Usable until
	Protect from sunlight
	Temperature limit

	Contains hazardous substances (see Chapter 1)
	Observe instructions for use (The round ISO 7010-M002 symbol on the packaging is blue)
	European Conformity Mark

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