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## Instruction for use (GB)

### 1. Product description

Megaseal N is suitable for use as a light-curing varnish for sealing plastic prostheses (PMMA), temporary crowns and bridges as well as tray plates made of light-curing tray material. The material is intended exclusively for use in the dental laboratory or for extraoral application (outside the oral cavity) in the dental practice.  
Megaseal N seals the surfaces and thus serves to protect against discoloration and delays the accumulation of plaque.

### 2. Application

Megaseal N is applied with the help of a brush to the clean & dry plastic surface in a thin film. Megaseal N has a slightly watery consistency and can therefore be spread easily. After a short drying time (solvent evaporation time approx. 30 s), the wetted surface is polymerized out in a commercially available light polymerization unit with UV or blue light in the light wave range between 320–500 nm (e.g. Megalight unit) for approx. 3–5 min. The polymerization time depends on the intensity of the device. High-performance devices (e.g. stroboscopes) allow a short polymerization time. However, this should not be less than 3 minutes.

### 3. Composition / Technical data

The light-curing surface sealer consists of a mixture of acrylates, methacrylates and bound nanoparticles.  
Composition: Multifunctional methacrylates, acrylates, nanoparticles, photoinitiators, stabilizers

### 4. Hazard information

Attention! Megaseal N contains methyl methacrylate and is flammable!

Boiling Point: 100°C  
Ignition temperature: 430°C

Methyl methacrylate and its vapor are highly flammable, cause skin irritation and can cause allergic skin reactions. It can irritate the respiratory tract.

Do not smoke, keep away from sources of ignition. Avoid skin contact in the unpolymerized state. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.

When handling chemical agents, only chemical protection gloves with the CE mark including a four-digit test number may be worn. The design of chemical protection gloves depends on the concentration and quantity of hazardous substances at the workplace. It is recommended to check the chemical resistance of the above-mentioned protective gloves for special applications with the glove manufacturer.  
Wear goggles/face protection when working. In case of contact with skin, wash immediately with plenty of water. Wear respiratory protection if there is insufficient ventilation.

### 5. Shelf life

23 years

### 6. Special storage and disposal instructions

Do not store above 25°C! Megaseal N contains methyl methacrylate and is flammable!  
Store in a well-ventilated place. This product and its container must be disposed of as hazardous waste. Contaminated packaging must be treated in the same way as the substance.

### 7. Dosage form and package sizes

20 ml bottle with brush insert item no. 2VLACK-N

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