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T-Fill Temporary Filling material of composite base

2. Composition:

Methacrylate-base (Urethandimethacrylat, Polymethylmethacrylat)
Dental-glass, initiators, stabilisators, pigments

3. Indication

T-Fill is used for temporary fillings and for temporary fixation of crown- and bridges.

4. Contraindications

So far unknown

5. Interactions

Application of eugenol containing material, specifically base liners, is not indicated, it will impair the polymerization.

6. Warning

Avoid eyes contact with non-polymerized material.

7. Application:

In all classes of cavities this may be kept to the minimum required for caries removal. Undercuts and dovetails are possible for fixation but not required. After the cavity is cut it has to be washed with water and dried with a gentle stream of air a desiccation is to avoid. Deep cavities or possible pulp exposure areas should be lined with a calcium hydroxide material.

T-Fill must applied with an instrument. The modulation follow according to the filling therapy.

After completed application, the restoration is light-cured for 20-40 s with Megalux unit or another halogen ore LED light unit. The final forming and the removal of excess composite are carried out with rotating finishing instruments.

T-Fill can be removed from cavity with instruments ore cutting bores.

Technical data

Total filler content	>	65,0 Gew%
Anorganic filler	>	40,0 Gew%

7. Storage stability

2 years

8. Storage and keeping instructions

dry, not above 25°C

9. Administration and package size

Syringe 4 g

Assortment 3 x 4 g

10. Date information published

04/2011

11. Manufactures and sales

MEGADENTA Dentalprodukte GmbH

D-01454 Radeberg, Germany

Internet: www.megadenta.de / info@megadenta.de